Instructions for Use English from page 03

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**Page 9 to 21 | Operation**

How to use Daysy.

**Page 22 to 30 | Technical notes and warranty**

What else you need to know.

# About Daysy

Daysy calculates a woman's daily menstrual cycle status by interpreting her basal body temperature. Daysy displays your menstrual cycle status using colored lights. Daysy is to be used to facilitate conception. Daysy is not a replacement for contraception, nor is it a medical thermometer.

## **Included in delivery:**

- Daysy
- Sensor cap
- Micro USB cable
- Instructions for Use
- Quick Guide

## — Precautions


- Before using your Daysy for the first time, please read these instructions carefully and keep them in a safe place for later reference.
- Before you start using your Daysy, you need to entirely discontinue hormonal methods of birth control.
- Daysy can better determine your cycle if you have regular cycles.
- Daysy is not suitable for use if you are undergoing hormonal treatment, or if you are using hormonal methods of birth control.
- Daysy does not protect you from sexually transmitted diseases.
- You should always read your daily fertility status from Daysy itself, not the app.
- Before using Daysy for the first time, charge the battery fully.
- Never open the housing.
- Never chew on the sensor.
- Make sure that the device or its individual parts are not swallowed.
- **WARNING:** The device must not be modified.
- **WARNING:** Avoid using Daysy at or less than 30 cm away from other electrical equipment, as this could result in improper operation.

- **WARNING:** Use of cables or equipment other than those specified in this document may cause electrical interference or lead to failure of Daysy.
- Do not expose your Daysy to high temperatures (e.g. boiling water).
- Do not expose your Daysy to direct sunlight for long periods of time.
- Do not expose your Daysy to liquids such as running water.
- Only take measurements in your mouth.
- Do not insert the measuring sensor too far into your mouth.
- Do not take measurements if the housing and/or sensor is broken.
- Do not take measurements if you have an injury in your mouth.
- Do not expose your Daysy to heavy mechanical loads.
- Only take measurements when the USB cable is NOT connected.
- Only charge your Daysy using a standard 5V USB port (IEC 60950-1 compliant) with a maximum output current of 1A and a maximum power of 4.5W with a cable shorter than 50cm. Do not take measurements while your Daysy is charging.
- If your Daysy causes irritation to your skin, we recommend you stop using it immediately and contact our Customer Support as soon as possible.

- Do not stick sharp objects into your Daysy.
- All serious incidents associated with this product should be reported to the manufacturer and the relevant government authority responsible in the EU member state.
- Your Daysy is intended for you alone. You should not share it with other women. If you do wish to pass it on to a friend, you will need to delete your data first and unpair Daysy in the app (if you use it).
- Only pass your Daysy on to another person with all instructions. You will also find the instructions and further useful information available for download at [daysy.me](https://daysy.me).
- As a user, you can use every feature of your Daysy. You can also clean the sensor yourself when your Daysy is switched off.
- Your Daysy is not a contraceptive.
- Keep Daysy away from children and pets.

# All the features at a glance\*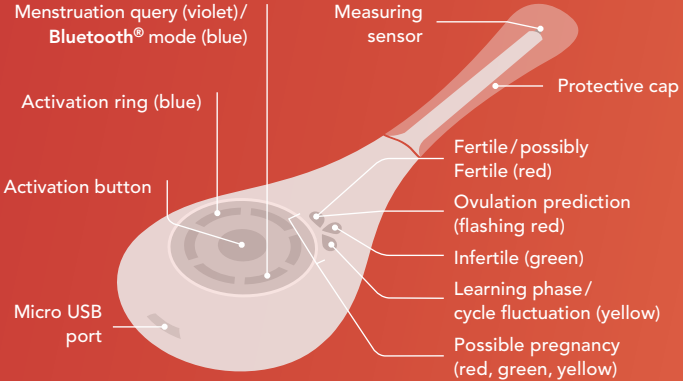

\* Page 13 shows you all the important display combinations

## — Step 1: Charge your Daysy

1. Connect your Daysy to the micro USB cable supplied in your package and the cable to a USB port. Make sure it is a 5V port that does not supply more than 1A current.
2. Press the activation button to see the charging status. The blue lights moving from bottom to top indicate Daysy is charging. Daysy will enter standby mode after 60 seconds but will continue charging.\*
3. All blue lights remain illuminated when Daysy is fully charged. Simply press the activation button once to check the status. (A low battery condition is indicated by all blue lights flashing.)
4. Disconnect the cable from your Daysy. You can now begin using your Daysy.\*\*

\* Do not measure or try to sync whilst Daysy is charging.

\*\* You can start measuring at any time in your cycle.

# Measurement indicators: What each indicator means

**Flashing violet light:** Your Daysy is asking if you are menstruating. Confirm that you are menstruating by holding down the activation button for 2 seconds.

**Violet light remains constantly illuminated:** Daysy confirms your menstruation input. If you have entered your menstruation incorrectly or by mistake, hold the activation button for 2 seconds until the violet light starts flashing again.

**Blue light rotating in a clockwise direction:** Your Daysy is measuring your temperature. Holding down the activation button for 2 seconds stops the measurement.

## — Step 2: Measure your temperature

Use your Daysy every day after waking up and before you get out of bed. Please be sure to sleep 3 consecutive hours immediately prior to measuring.

1. Remove the protective cap and briefly press the activation button once. Your predicted fertility status will be displayed.
2. Press the button a second time and the activation ring will flash in a clockwise direction. You can now take a measurement.
3. Place the sensor as far back as is comfortable under the right or left side of your tongue and then close your mouth.
4. "Beep-beep" means that the measurement phase has successfully completed.\*
5. The confirmed fertility status displayed after measurement is valid until midnight. You can view your fertility status again later in the day by pressing the activation button once briefly. The next day, your Daysy switches to the predicted fertility status before your measurement.

*\* If you hear a "grumble-buzz" sound instead of a "beep-beep", it means that the measurement was unsuccessful, and you will need to try again.*

# Fertility indicators: What each indicator means

Fertile/possibly fertile

Ovulation prediction

Infertile

Possible pregnancy

Learning phase/  
cycle fluctuation  
(potentially fertile)

Highly probable  
pregnancy

## — Step 3: Enter your menstruation

If the violet **“Menstruation”** light is flashing in the activation ring, it means that Daysy is asking for your menstruation input.\*

- **If you are menstruating:** Press the activation button until the violet light illuminates without flashing and you hear a “beep”. Confirm you are menstruating for each day of active bleeding and flow for a minimum of 3 consecutive days. Always confirm menstruation, even if you did not take your temperature that day.
- **If you are not menstruating:** You do not have to do anything, just take your temperature.
- **If you are menstruating, but Daysy is not showing the flashing violet light:** Press the activation button until the violet light is constantly illuminated.

*\* Menstruation can be edited for up to 3 past days. Please enter and edit menstruation on the device itself. For instructions, watch our tutorials at [daysy.me/tutorials](https://daysy.me/tutorials).*

# DaysyDay: Better informed with the Daysy app

DaysyDay provides valuable insights to your cycle and additional viewing and tracking features, as well as a Partner App.

Turn on **Bluetooth®** on your mobile phone, open DaysyDay and select the **Bluetooth®** button. Follow the on-screen instructions to turn Daysy on and press the Daysy activation button twice briefly. A blue light will light up on your Daysy followed by a brief melody. If the blue light stays lit, it means the connection was successful.


**Flashing blue light:**  
Daysy is connecting.

**Continuous blue light:**  
Daysy is connected.

# Temperature and fertility: The course of your cycle

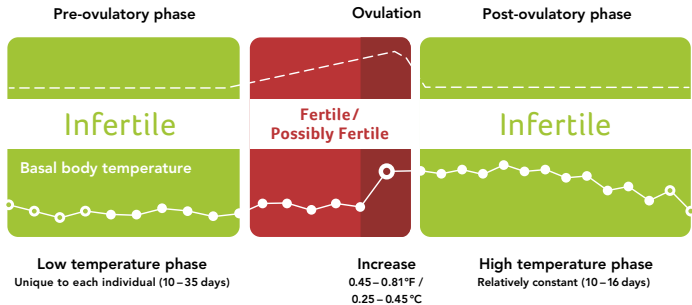# Important: How to you use your Daysy correctly

- Make sure the first thing you do is take your temperature every morning before you get out of bed. For the optimal use of Daysy we recommend that you sleep at least 3 consecutive hours before measuring. If you forget, and have already gotten out of bed, do not proceed to take your temperature.
- If you are menstruating, enter this input accordingly. It is important to begin with the first day of active red blood flow.\*
- The more regularly you take your temperature, the faster Daysy will get to know your cycle.
- When you start using your Daysy, you will initially see a lot of yellow and red days. Over the course of the next 3–4 cycles, Daysy learns your unique menstrual cycle rhythm.

\* **Important:** *The first bleeding after stopping the pill or other hormonal contraceptive methods is a hormonal withdrawal bleeding and not a menstruation. DO NOT enter this bleeding as menstruation.*

## — Cleaning

- Clean Daysy before and after every use.
- Use a damp cloth to carefully clean the sensor on your Daysy. You can also use a little alcohol or soap if you wish. Since your Daysy is not waterproof, make sure that only the neck of the sensor, and not the whole appliance comes into contact with water. Do not clean under running water.

## Battery

- A single battery charge will last 1–2 months, depending on how often you use your Daysy. To indicate that the battery is nearly empty, the activation ring on your Daysy will flash at frequent intervals. Make sure you charge the battery within a week. With normal use, the battery will still charge to 80% of its capacity after 500 charging cycles.
- **Note:** You should not take measurements while your Daysy is charging.

# Error messages / Problems during use

## ***My Daysy does not respond when I press the activation button.***

Please charge your Daysy and try activating her again before contacting our Customer Support.

## ***The activation ring on my Daysy flashes blue at short intervals.***

Please charge the battery on your Daysy.

## ***My Daysy fails to start taking a measurement.***

- Remove the USB cable before taking a measurement.
- If your Daysy makes a “beep-beep” sound, it means you have already successfully taken a measurement today.

*For more information, read our FAQ at [daysy.me](http://daysy.me) or contact our Customer Support.*

---

***My Daysy ends the measurement with a “grumble-buzz” instead of a “beep-beep”.***

The measurement took too long, was interrupted, or was unsuccessful.

Try following these tips:

- As before, place the sensor as far back as is comfortable under the right or left side of your tongue.
- Close your mouth.
- Do not remove the your Daysy from your mouth until you hear the “beep-beep”.
- Daysy may not accept the measurement if you have a fever.  
Stop taking measurements if you are sick.
- If the sensor gets dirty, simply clean it with a damp cloth.

***The red indicator lights up and the activation ring flashes at short intervals.***

Something is wrong with your Daysy. Please contact our Customer Support.

## ***I won't use my Daysy in the next few months.***

### ***What do I need to do?***

If you do not intend to use your Daysy for a long period, clean her, charge the battery fully, and store her in a cool, dry place. Make sure to charge your Daysy every 2–3 months.

### ***How does my Daysy communicate with me?***

- “Beep-beep”: Daysy is happy. The measurement was a success.
- “Grumble-buzz”: Daysy is unhappy. The measurement was not a success.
- “Beep”: Daysy beeps when you press the activation button.
- Melody: Daysy has just activated **Bluetooth®**.

*For more information, read our FAQ at [daysy.me](https://daysy.me) or contact our Customer Support.*

# — Technical data

## **Key features**

**Name and model:** Daysy **Power supply:** Internal power source

**Contains FCC ID:** WAP2001 **IC:** 7922A-2001 **USB port:** USB 2 Micro-B

## **Compatibility**

Compatibility at date of manufacture. Current compatibilities available on the website and app store descriptions.

**iOS:** Version 12.0 and higher **Android:** Google Android version 5.0 and higher

**Bluetooth®:** Version 4.0 and higher

**Dimensions:** H/W/D 13/2.2/4.8 cm **Weight:** 38g **EAN Code:** 7629999 054073

## **Technical Information**

You need a **Bluetooth®**-enabled smartphone or a tablet running iOS 12.0/Android 5.0 or higher to be able to use the DaysyDay app.

**Bluetooth®:** Bluetooth® 4.1 **Frequency band:** 2400 – 2480 MHz

**GFSK modulation:** FHSS **Transmitting power:** 0 dBm

# Indications and contraindications

**Indications:** Daysy is a menstrual fertility tracker. For optimal use of Daysy, cycle lengths of 19–40 days are the ideal requirement.

**Contraindications:** In the following cases, you should **not** use Daysy:

- You suffer from a condition in which pregnancy could be dangerous for you, e.g. your blood pressure is too high or too low, you are suffering from heart disease, etc.
- Health risks exist during pregnancy for your unborn child, e.g. if you are suffering from drug or alcohol addiction, or if you are taking medication that may cause birth defects.
- If you are undergoing hormonal treatment, or if you are using hormonal methods of birth control.
- In case of irregular bleeding. This can interfere with the prediction of your fertility or render it unusable.
- Daysy is not a medical thermometer and should not be used to measure body temperature as an indication of fever or illness.

## — Note on electromagnetic compatibility

**Electrical medical appliances are subject to special precautions regarding electromagnetic compatibility (EMC).** Daysy is intended for non-commercial home use only.


**Portable and mobile RF communication appliances may affect medical electrical equipment.** Daysy meets the requirements of EN 60601-1-2 for electromagnetic compatibility. Details of this measurement data can be obtained from our Customer Support or from [daysy.me](http://daysy.me).

## Target group profile

Women aged 18 or older, who can read and write.

# Device Label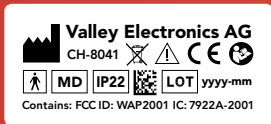

The following figure shows the device label that is attached to the back of your Daysy. The label is shown enlarged for improved readability.

# General information

**Manufacturer**

**Read these instructions carefully**

**Attention**

**CE label**

**Medical device**

**LOT number/  
manufacturing date**

**EU Authorised  
Representative**

**WEEE:** Please dispose of this appliance at public recycling depots for electrical appliances. Local laws/regulations prohibit disposing of this appliance with your general domestic garbage.

**Type-BF application parts**

**IP code:** Protected against solid foreign bodies with a diameter of 12.5 mm and splash-proof when the housing is tilted at 15°.

**Patent number:** US 8,998,822  
**U.S. Patent** D865,542

# Storage and operating conditions

**Temperature limit:** This device can be stored at temperatures between  $-4^{\circ}\text{F}$  and  $113^{\circ}\text{F}$  /  $-20^{\circ}\text{C}$  and  $45^{\circ}\text{C}$ .

**Temperature limit:** This device can be used without restriction at temperatures between  $41^{\circ}\text{F}$  and  $104^{\circ}\text{F}$  /  $5^{\circ}\text{C}$  and  $40^{\circ}\text{C}$ .

**Humidity limit:** This device can be stored without restriction at a relative humidity up to 93 %.

**Humidity limit** This device can be used without restriction at a relative humidity of between 15 % and 93 %.

**Pressure limit:** This device can be stored at pressures of between 700 hPa and 1060 hPa.

**Pressure limit:** This device can be subjected to pressures of between 700 hPa and 1060 hPa.

## — Warranty information

Daysy has been manufactured with the utmost care using premium-quality, state-of-the-art materials. Each Daysy is subjected to numerous controls and inspections prior to delivery.

1. We guarantee the original purchaser that Daysy is free of defects in materials and workmanship during the warranty period.
2. The warranty period begins on the day the new Daysy is sold to the end user by an authorized dealer.\* The warranty is valid for two years.
3. We undertake to either repair a defective device or replace it with a new one. Further claims, and, in particular, claims for losses and damages, including consequential damages, are excluded. The warranty does not cover indirect damages.

*\* Please note that our limited warranty applies only if you purchased your Daysy from an authorized Valley Electronics dealer.*

4. The warranty does not cover any damage caused by improper handling, especially by opening the device, or by other external influences.
5. Warranty claims are only valid if the user indicates the defect immediately in writing within two weeks after such a defect is discovered. Once the defect is discovered, the user should no longer use the Daysy, but return it to the authorized distributor with a copy of the original purchase invoice via insured delivery.

If you have any questions or require further information, please do not hesitate to contact your Daysy dealer.

# — Overview: What do the indicators mean?

Status Indicator			M light	Ring	Meaning
Red	Green	Yellow	Violet	Blue	
Illuminated	–	–	–	–	Fertile / possibly fertile
Flashes	–	–	–	–	Ovulating (prediction)
–	Illuminated	–	–	–	Infertile
–	–	Illuminated	–	–	Learning phase or cycle fluctuation
Illuminated	Illuminated	Illuminated	–	–	Highly probable pregnancy
Flashes	Flashes	Flashes	–	–	Possible pregnancy
–	–	–	Flashes	–	Daysy asks for your menstruation input
–	–	–	Illuminated	–	Menstruation confirmed for this day
–	–	–	–	Rotates	Temperature measurement
–	–	–	–	Flashes	Charge battery
–	–	–	–	Rising	Battery charging
–	–	–	–	Illuminated	Battery charged
–	–	–	–	1 light flashes	<b>Bluetooth®</b> enabled, but no connection
–	–	–	–	1 light illuminated	<b>Bluetooth®</b> enabled, connection is established

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Valley Electronics AG
Valley Electronics LLC
M.I.S.S. GmbH